

{Hands and Arms} Microdermabrasion Treatment

Cleanse:

- Gentle Gel Cleanser: “Removal of debris without irritation”

Toner:

- Salicylic Toner: “BHA/AHA treatment toner”

Microdermabrasion:

Settings:

- Set vacuum gauge at 15; crystal flow at 12 o'clock. Proceed with general peeling to the entire skin surface or area of concern

Remove Crystals:

- Remove with a hot towel and or Gentle Gel Cleanser

Tone:

- Follow with Instant Soothing Toner

Treatment (Optional)

- May incorporate Professional Peeling System. Please see Product Manual for protocol

Nourish:

- Vita C Peptide Serum with advanced LSS™ delivery
- Moisture Booster + hyaluronic acid with LSS™ delivery
- Ultra Performing Complex + vitamins A, C, E with LSS™ delivery

Mask:

- Revitalizing Mint Mask; leave on for 10-15 minutes then remove with tepid towel

Restore:

- Moisture Booster + hyaluronic acid with LSS™ delivery
- Ultra Performing Complex + vitamins A, C, E with LSS™ delivery