

Oxygen Infusion Treatment System

Sensitive

Cleanser:

- Gentle Gel Cleanser + natural phospholipids
- Gentle Milk Cleanser + reaffirming nutrients

Tone:

- Instant Soothing Toner + natural phospholipids

Treat:

- Micro-dermabrasion: Aluminum Oxide and or Organic Grains
 - Setting: Vacuum gauge at level 10-12; crystal flow setting 2o'clock. Proceed with your exfoliation treatment.
 - Remove crystals
 - Use toner to remove any residual
- Enzyme Gel + papaya complex
 - Use face mask brush and apply to the entire face and manipulate for 30 seconds
 - Then apply steam for 5 minutes or less
 - Hot towel to the entire face for 1 minute or less
 - Remove Enzyme
 - Apply toner to remove residual

Oxygen Infusion System:

1. Apply a thin layer of Sensitive; Oxygen Infusion Treatment to the entire face and neck
2. You may manipulate with finger tips for 30 seconds
3. Spray oxygen treatment + activator over the entire face and neck area
 - a. Forehead
 - b. Temple to Cheek area
 - c. Mandible (Jaw line)

- d. Transitioning to neck area
4. May manipulate each area with light massage movements after spraying activator per area
5. Allow product to remain on the skin for 3-5 minutes, you may see a foaming oxygen infusion reaction
6. Remove with tepid water, by using gauze

Soothe: Apply Instant Soothing Toner + natural phospholipids

Treat: May use the following

LED Light Therapy: Use Red LED Light Therapy

Nourish:

- Vita C Peptide Serum + advanced LSS™ delivery
- Moisture Booster + hyaluronic acid with LSS™ delivery

Replenish:

- Vital recovery Concentrate + healing botanicals with LSS™ delivery

Restore:

- Ultra Performing Complex + vitamin A, C, E with LSS™ delivery

Protect:

- Anhydrous Barrier SPF 20 + oil-free concentrate